

## Change table operating instructions

**The Safe Working Load for the bench is 200 kgs, this table is not to be used by persons above this weight.**

The following instructions **MUST** be complied with each time the change table is used.

1. Gently unfold the table from the upright position.
2. Do not raise or lower the table whilst in the folded up position.
3. Nothing should stand in the way of the table when in motion.
4. Press the  $\wedge$  button to raise the table and the  $\vee$  button to lower the table.
5. Cover the table with disposable paper cover.
6. Raise the side safety rail when the person has been placed on the table. To unlock the side safety rail, pull out pin on right side and unfold.
7. After use dispose of paper table cover.
8. Fold the side safety rail back to the flat position after use. Use pin to secure in place.
9. Fold the table up into the upright position.

### Remember


- This table is not to be played with at any time. The table is to be used for the purpose for which it is designed.
- Children are not to operate the bench at any time.
- If the table is out of order, it must not be used.

## Ceiling hoist operating instructions

**The maximum Safe Working Load of this hoist is 180kg reduce weight recommended by manufacturer. This hoist is not to be used by persons above this weight.**

If the red light is on, do not use this hoist. If there is any doubt about the safety of using this hoist, it **MUST NOT** be used.


1. Use the control to raise or lower the spreader bar to the required height.
2. Attach the hooks of the sling in the lifting hanger's hooks. Check the sling is securely attached before lifting the person. (Fig. 1a & 1b)

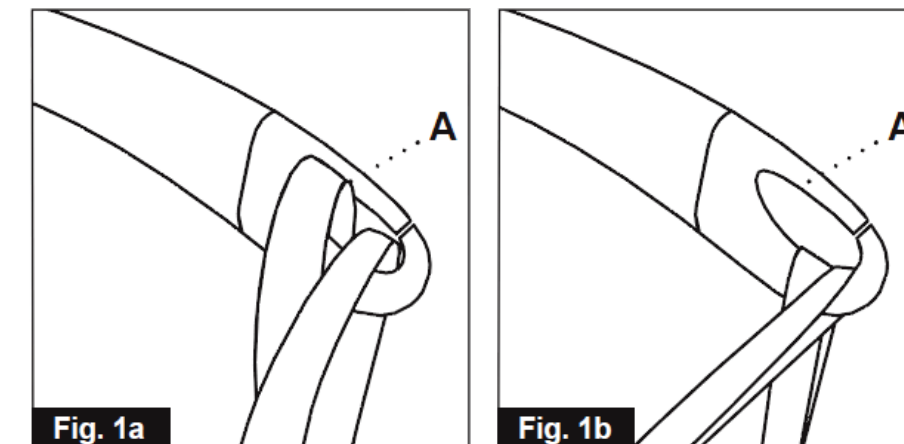

3. Check nothing will stop the hoist from lifting the person.

4. Use the control buttons to raise the person.


- 1 – Lift
- 2 – Lower

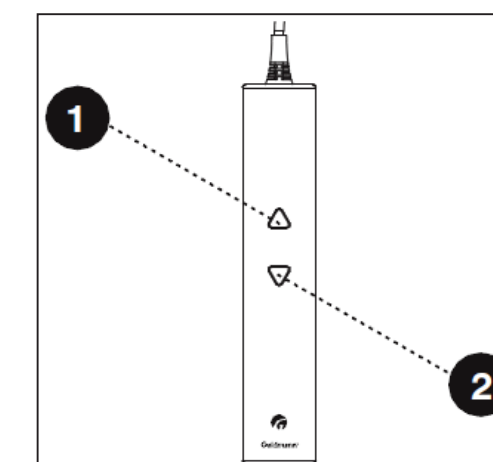


5. After use, return hoist to charging station



6. Emergency lowering strap can be used to lower a person in an emergency. Pull the red strap (once/continuously).

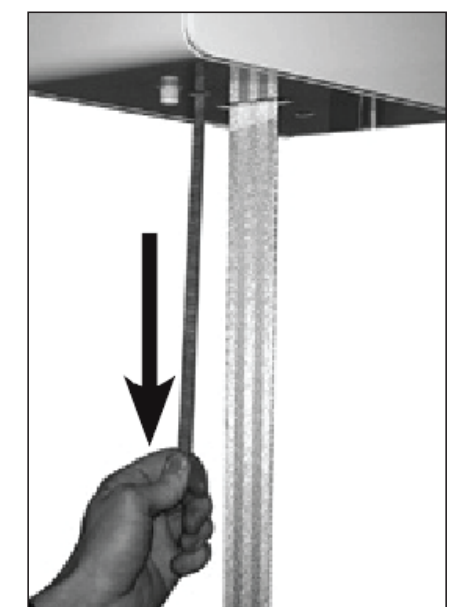

Once activated the hoist will not operate until the yellow button is manually pressed in.

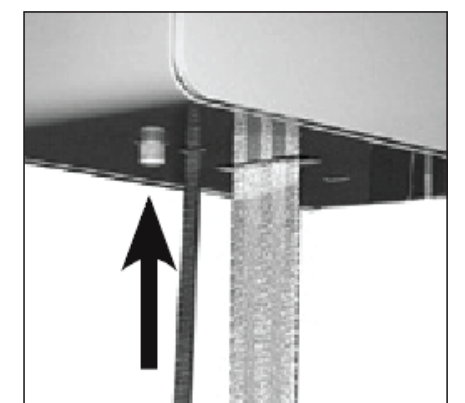


7. Warning lights

Green light – hoist is ready to use

Yellow light is on – hoist battery is low or flat. Hoist must be charged before using.

Red light – do not use

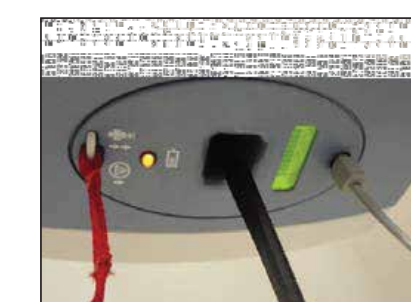

**To report a fault or a maintenance issue, contact the City of Melville on 1300 635 845**